

*Strong Dental Inc. is doing something new...*



The  
**SUAD™**  
Device

*now comes packaged with...*

The  
**MORNING**  
Repositioner

It has long been known that use of mandibular repositioning devices carry with them significant incidence of forward posturing of the mandible during the day and in some cases may lead to a Class III occlusion over time.

Regardless as to why this happens, one thing is certain; patients need to perform some sort of isometric contraction of the musculature as quickly as possible after removing their appliance to help avoid permanent advancement of the mandible.

With today's fast paced lifestyle, Strong Dental feels it is essential to have a device that is not only effective, but easy to use. The Morning Repositioner is a hands free device that a patient can use for just 10-15 minutes while going about their morning routine. Our belief in the importance of The Morning Repositioner is so strong; we are now including it with every new SUAD™ Device we make.



The SUAD™  
Device is  
**FDA**  
APPROVED

*Improving Quality of Life Through Better Sleep.™*

**Special  
Offer!**

Strong Dental Inc. is so confident you and your patients will benefit from this Morning Repositioner, we would like to offer you your first one free.

See the back of this flyer for more information.

*Introducing...*

# The **MORNING Repositioner**

## Strong Dental Inc.'s Bite Deprogrammer

The objective of the "Morning Repositioner" is to allow the mandible to be able to "float or pivot" while trying to reposition itself. The raised blocks are placed onto the bicuspids directing all forces straight down towards the roots and not laterally or forward onto the anterior teeth. When closed, the blocks should be the only area of contact, thereby giving the pivot effect. The molars should not contact during clenching so that the muscles can get the full effect of the isometric exercise. The inner parts of the blocks are made out of a soft material allowing them to compress thereby equalizing the pressure from side to side. The whole over-layer of the device is made out of a laminated material developed specifically for night splits thereby giving greater wearability.

Patients are instructed to place the "Morning Repositioner" on their lower dentition and keep it in place for approximately 10 - 15 minutes while going about their morning routine. It is recommended that the patient do a series of clenches on the raised blocks; concentrating on keeping their mandible as retruded as possible. This retrusion can be achieved by asking the patient to try to make the blocks on the "Morning Repositioner" touch their upper second molars. Patients are instructed to clench or chew up and down as often as possible and hold for as long as is comfortable.

The "Morning Repositioner" is included with every new SUAD™ Device, fabrication at no charge to the practitioner. For a "Morning Repositioner" requested on its own, the lab fee is \$49.00US (\$55.00CDN).

Strong Dental Inc. is so confident you and your patients will benefit from this Morning Repositioner, we would like to offer you your first one free!



To request shipping supplies and lab scripts, or to receive more information, please contact:

**Strong Dental Inc.**  
**1-800-339-4452**

**Canadian Shipping/Receiving**  
33 Princess St. - Suite 403  
Leamington, ON, Canada  
N8H 5C5

**U.S. Shipping/Receiving**  
3430 E. Jefferson Ave. -  
Suite 307  
Detroit, MI, U.S.A. 48207

[info@strongdental.com](mailto:info@strongdental.com) / [www.strongdental.com](http://www.strongdental.com)

Federal law restricts this device to sale by or on the order of a physician, dental practitioner, or professional trained in sleep disorders.